Tube Squeezer

VM917

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Tube Squeezer from Aidapt. The Tube Squeezer is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

The Tube Squeezer makes it easy to put pressure on toothpaste and similar tubes, squeezing out the contents. The Tube Squeezer is ideal for one-handed use and for those with arthritis, as well as thrifty or eco-friendly users keen to reduce waste by ensuring that tubes are completely empty before throwing them out.

Suitable for most products stored in tubes, whether in the bathroom, kitchen or utility room.

INSTRUCTIONS FOR USE

- Ensure that the surface on which you will be positioning the Tube Squeezer is clean and dry.
- Position the Tube Squeezer as desired then press down to secure it to the surface with the built-in non-slip suction base.
- Slide the 'key' over the end of the tube, then position the key into the base and twist to squeeze the tube.
- The suction base is designed to hold the Tube Squeezer in position while the tube is being squeezed with the 'key' using just one hand if necessary.

CLEANING

Clean your Tube Squeezer using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you rinse then wipe dry the equipment after cleaning.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

Usage and Maintenance Instructions

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Altai Germany (Altai Europe Ltd.)