




## **VM310 – Analog Talking Clock**

## BATTERY INSTALLATION

- The Clock is powered by 2 x AA batteries (Not Included)
- On the base of the clock you will find the battery compartment, open compartment and install the batteries, please ensure the correct polarity is used once inserted then replace cover.

## INSTRUCTIONS OF USE

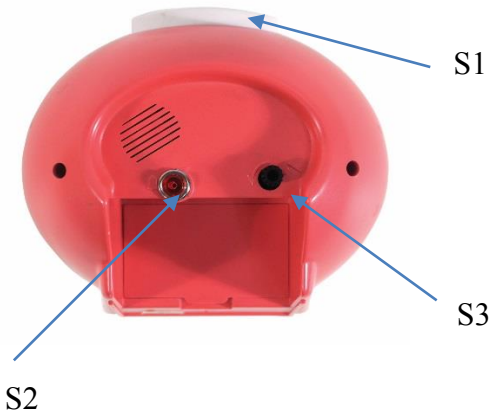

There are 2 controls on the reverse:

- (S1) Press to Speak & Set Button
- (S2) Time/Alarm set button
- (S3) Dial to control the analog hands

### **To Set the Talking Time, Date and Alarm**

- You have approx. 10 Seconds to change any value and then the clock will beep to indicate it has returned to normal.
- To Set Hour: Press S2 to set hour (the clock will repeat “Set hour”, then press S1 to adjust hour till the correct time is set.
- Set Minute: Press S2 till the chosen “set minute”, then press S1 to adjust minute
- Set Year: Press S2 till the chosen “set year”, then press S1 to adjust year
- Set Month: Press S2 to set month, then press S1 to adjust month
- Set Day: Press S2 to set day, then press S1 to adjust day

- Set Alarm Hour: Press S2 to set alarm hour, then press S1 to adjust hour
- Set Alarm Minute: Press S2 to set alarm minute, then press S1 to adjust minute
- Set Alarm: Press S2 to set alarm, then press S1 to turn alarm On/Off In setting mode, clock will return to normal time mode if no buttons are pressed for 5 seconds

### **To Use**

Press S1 once to hear the time spoken

Press S1 twice to hear the date spoken

Once Alarm Starts Press S1 to silence the alarm.

Use button S3 to adjust the hands to the correct time on the clock

## **SERVICE WARRANTY**

- Aidapt guarantees the product free from defects in materials and workmanship for a period of twelve months.
- The product you buy may sometimes differ slightly from illustrations. This warranty is in addition to, and does not affect, your statutory rights.
- Our guarantee is administered by our retailers.
- If your product arrives damaged, you must contact the retailer from whom you bought it. The retailer's contact details will be on the invoice that arrived with the product, or on the e-mail you received when you placed the order. Do not contact Aidapt, only your retailer can arrange a replacement.
- If your product fails within the guarantee period, and needs to be repaired or replaced, you must contact the retailer from whom you bought it.
- The retailer's contact details will be on the invoice that arrived with the product, or on the

e-mail you received when you placed the order.

- If you have just received your product and require technical help in using it, please call our Help Desk on (0845) 459 4816
- If you have any other technical queries about the product, please call our Help Desk on (0845) 459 4816.

Aidapt Bathrooms Ltd.,  
Lancots Lane, Sutton,  
St. Helens, Merseyside.  
WA9 3EX

Website: [www.aidapt.co.uk](http://www.aidapt.co.uk)